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1 Remove the old battery

Start with the meter off.

Open the battery door and pull up on the battery ribbon. The white ribbon is for the meter battery, and the black ribbon is for the backlight battery.

2 Insert the new battery

With the “+” side facing up toward you, place the battery in the compartment within the fold of the ribbon. Push the battery until it snaps into the battery clasp. Insert the two battery door tabs into the matching holes on the meter and push down until you hear the door click into place.

3 Check your meter settings

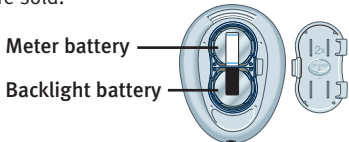
Removing the meter battery will not affect your stored results. However, you may need to re-set your meter settings. See your owner’s booklet.

4 Dispose of batteries according to your local environmental regulations

⚠ CAUTION: If you test at the low end of the operating range (43°F) and your glucose is high (over 180 mg/dL), the reading on your meter may be lower than your actual glucose. In this situation, repeat the test in a warmer environment with a new test strip as soon as possible.

Replacing batteries

Your OneTouch® Ultra® 2 Meter uses two 3.0 Volt CR 2032 lithium batteries (or equivalent)—one for the meter and one for the backlight. Replacement batteries can be found in most stores where batteries are sold.

⚠ WARNING: To avoid a possible shock, do not change either battery while the meter is connected to a computer with the OneTouch™ Interface Cable.

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What to do if test results are too high or too low

⚠ CAUTION: Low glucose results

If your test result is lower than 70 mg/dL or is shown as LOW GLUCOSE, it may mean hypoglycemia (low blood glucose). This may require immediate treatment according to your healthcare professional’s recommendations. Although this result could be due to a test error, it is safer to treat first, then do another test.

You may also get false low results if you are severely dehydrated.

⚠ CAUTION: High glucose results

If your test result is higher than 180 mg/dL, it may mean hyperglycemia (high blood glucose). If you are uncertain about this test result, consider re-testing. Your healthcare professional can work with you to decide what actions, if any, you should take if your results are higher than 180 mg/dL.

If your meter displays HIGH GLUCOSE, you may have a very high blood glucose level (severe hyperglycemia) exceeding 600 mg/dL. Re-check your glucose level. If the result is HIGH GLUCOSE again, obtain and follow instructions from your healthcare professional without delay.

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4 Get a drop of blood

Before testing, wash your hands and the puncture site. Rinse and dry. Use the lancing device and a new lancet to get a blood drop **of at least one microliter (● actual size).**

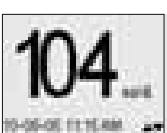
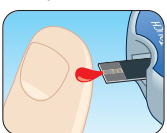
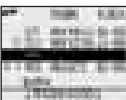
5 Apply the blood drop and read the result

Touch and hold the drop of blood to the narrow channel in the top edge of the test strip.

Keep holding the drop of blood to the top edge of the test strip until the confirmation window is full. If your sample does not fill the confirmation window on the first try, do not add more blood to that strip. Instead, test again with a new strip.

After your meter counts down from 5 to 1, your blood glucose level appears on the display, along with the unit of measure, and the date and time of the test.

⚠ WARNING: If mg/dL does not appear with the test result, contact Customer Service.

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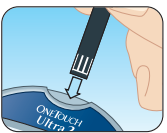
Before testing, read the OneTouch® Ultra® 2 Owner’s Booklet carefully.

How to test

1 Check the code on the test strip vial

2 Insert a test strip

Start with the meter off. Insert the test strip into the test port as shown. Make sure the three contact bars are facing you. Push the strip in as far as it will go. Do not bend the strip.

3 Match the code displayed on the meter with the code on the test strip vial

If necessary, press **▲** or **▼** to change the code on the meter to match the code on your test strip vial.

⚠ CAUTION: This step is essential to obtaining accurate results.

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